

Gambro

cleancart

CARTRIDGES

With **CleanCart A** and **CleanCart C** cartridges, regular dialysis machine maintenance and disinfection assure proper dialysis machine function and requires less intensive machine operation from clinical staff.

- Less space for storage and less heavy lifting for staff compared to liquid disinfectant
- Minimizes risk of personal injury and material damage when handling chemical agents
- Easy to streamline disinfection combining heat disinfection with decalcification and cleaning
- Removes organic deposits, fats and proteins with weekly use of **CleanCart A** cartridge¹
- Removes precipitated calcium and magnesium salts with daily use of **CleanCart C** cartridge¹
- Recyclable polypropylene container for less environmental impact compared to the liquid disinfectant

Dialysis machine maintenance and disinfection becomes a convenient and simple operation, so you can make the most of your time with patients.



Gambro **CleanCart** cartridge – specifications

CLEANCART A

The **CleanCart A** cartridge contains 13 grams of anhydrous sodium carbonate powder. The **CleanCart A** cartridge must be used in combination with the heat disinfection program to clean the dialysis machine fluid path. When circulated throughout the fluid path, the sodium carbonate solution removes organic deposits, fats and proteins¹

CLEANCART C

The **CleanCart C** cartridge contains 32 grams of citric acid anhydrous powder. The **CleanCart C** cartridge must be used in combination with the heat disinfection program to both decalcify and disinfect the dialysis machine fluid path simultaneously. When circulated throughout the fluid path, the citric acid solution removes precipitated salts of calcium and magnesium¹

TECHNICAL DATA

	CleanCart A	CleanCart C
Action in combination with the heat disinfection program	Cleaning of the fluid path from organic deposits, fats, and proteins	Disinfection & decalcification of the fluid path
Cartridge content	13 grams	32 grams
Total cartridge weight	Approx. 32 grams	Approx. 50 grams
Cartridge and label material	Recyclable polypropylene (PP)	Recyclable polypropylene (PP)
Recommended usage	Refer to operator's manual for corresponding dialysis machine	Refer to operator's manual for corresponding dialysis machine
pH (prepared solution)	Approx. 11	Approx. 2
Residual test needed	No	No
Biodegradable content	Yes	Yes
Storage conditions	Store below +30°C	Store below +30°C
Shelf life	24 months from date of manufacture	24 months from date of manufacture
Product code	114010	114011
Packaging	Box of 33 cartridges	Box of 33 cartridges
Box dimensions	279 x 247 x 81 mm	279 x 247 x 81 mm
Total box weight	1200 grams	1800 grams

1. Operator manual